

topsshelforthopedics.com  
Phone: 866.592.0488 • Fax: 209.834.8832  
Tracy, CA 95304  
1851 East Paradise Road, Suite A



## Wrist Sleeve

### Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.



# Wrist Sleeve

## Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.

**Indications:** To provide support of the wrist associated with wrist sprain/strain.

### **Application Instructions:**

1. Insert hand into opening and position buckles so they are on the top surface of the wrist.
2. Position sleeve and then insert strap through buckle closest to the hand first; secure to self with velcro-type closure material.
3. Secure second strap in similar manner to the first.
4. Adjust as necessary for comfort.

**Use & Care:** Hand wash in cold water using mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

**Warning:** This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

**Warning:** Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene.

**Warranty:** Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months from the date of purchase.

**Caution:** Federal Law (USA) restricts this device to sale by or on the order of a licensed healthcare professional.