

topsselforthopedics.com  
Phone: 866.592.0488 • Fax: 209.834.8832  
Tracy, CA 95304  
1851 East Paradise Road, Suite A



## Lunar Air/Ankle Walker

### Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of the brace.




# Lunar Air/Ankle Walker

## Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of the brace.

**Indications:** Soft tissue injuries of the lower leg, acute ankle sprains, stress fractures of the lower leg, stable fractures of the foot and ankle, stable delayed union or non-union fractures of distal tibia and fibula, achilles tendon repairs.

### Application Instructions:

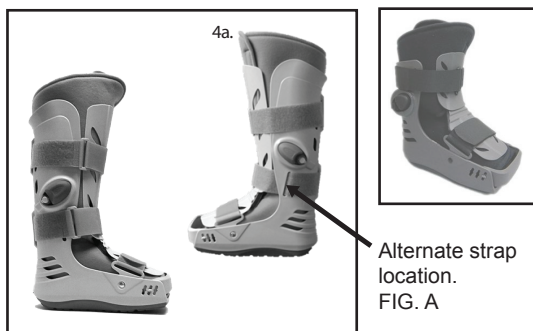

1. Unfasten straps, remove front panel and open liner.
2. While seated, place leg in brace and position heel at the back of the brace. Wrap liner around the leg and foot and secure.
3. Replace the front panel.
4. Fasten the straps from the bottom to the top.
- 4a. For additional control of the ankle, use the alternate location strap shown in Fig. (A).
5. Adjust air bladder compression by pushing the bulbs on either side of the boot. To deflate the air bladders, open the cap of the smaller button next to the bulbs and push the button to release air.

### Use and Care:

Hand wash the liner with mild soap & water & lay out to dry. Do NOT machine dry.

### Notes:

For single patient use only. Latex free.

### Warning:

This product is to be used under the supervision of a medical professional. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

### Warranty:

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of 6 months from the date of purchase.

### Caution:

Federal Law (U.S.A.) restricts this device for sale by or on the order of a licensed healthcare professional.