

APPLICATION AND FITTING INSTRUCTIONS

1. Position elbow in the pocket of the sling.
2. For Arm Slings with a thumb loop, place thumb in loop if desired.
3. Place strap over opposite shoulder, feed through the D-ring and secure to itself. Hand should be slightly elevated above the elbow when secured.
4. (For Universal Size) The sling length may be shortened by folding the open end inside itself and securing the strap closure. For strap length adjustment use slide buckle.
5. For Arm Slings with a shoulder pad, slide pad onto strap and position in desired location.
6. For Shoulder Immobilizers follow steps 1-3 above, then attach waist strap.

CARE INSTRUCTIONS

Hand wash in cold water using mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

WARNING

If you experience pain, swelling, or sensation changes as a result of wearing this brace, consult your healthcare provider immediately.

Federal Law (USA) restricts this device to sale by or on the order of a licensed healthcare professional.

Arm Sling

Instructions For Use

Indication:

Immobilization & support of the
shoulder and elbow joints

Before applying this upper extremity support, please read these instructions completely and carefully. Correct application is important for proper function.

Manufactured By:

