

# **Industrial Back Support**

## **Instructions For Use**

Indication:  
Immobilization & Support of the Lower Back

**Please read these instructions completely and carefully before applying this brace.  
Correct application of the brace is important for proper function.**

Manufactured By:



## APPLICATION AND FITTING INSTRUCTIONS

- 1.** If suspenders are included and not sewn to support, attach with buckles in front and rear. Place straps over shoulders.
- 2.** Center support on lower back. Pull support around waist, over the hips and secure in front forming a “V” below the navel.
- 3.** Adjust suspenders for comfort & fit.
- 4.** When performing physical tasks in supports with double side pulls, grasp outer straps with both hands, stretch forward and secure. Upon completion of task, release outer straps and return to neutral position.

### Sizing Chart:

(Measure waist for male, hip for female)

XS	22” – 24”
SM	24” – 30”
MD	30” – 36”
LG	36” – 42”
XL	42” – 50”
2XL	50” – 56”
3XL	56” – 64”
4XL	64” – 72”

## CARE INSTRUCTIONS

Hand wash in cold water using mild soap. Rinse thoroughly and air dry.  
If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

## WARNING

If you experience pain, swelling, or sensation changes as a result of wearing this support, consult your healthcare provider immediately.  
(Never wear unfastened)

Federal Law (USA) restricts this device to sale by or on the order of a licensed healthcare professional.