

topsselforthopedics.com  
Phone: 866.592.0488 • Fax: 209.834.8832  
Tracy, CA 95304  
1851 East Paradise Road, Suite A



# Elbow Sleeve

## Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.



# Elbow Sleeve


## Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.

**Indications:** For general support and compression of the elbow.

### **Application Instructions:**

Grasp edges of the sleeve at the larger opening. Bring sleeve onto the arm so that the elbow is centered within the sleeve.

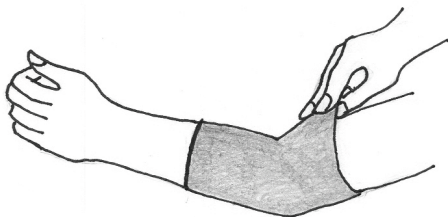

### **Use and Care:**

Hand wash in cold water using mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, residual soap may cause skin irritation and material breakdown.

### **Warning:**

This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

### **Notice:**

This product does not contain latex.

### **Warranty:**

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months from the date of purchase.

### **Caution:**

For single patient use only.