



# Universal Tendonitis Strap

## Instructions For Use


Please read these instructions completely and carefully before applying this brace. Correct application of the brace is important for proper function.




## UNIVERSAL TENDONITIS STRAP APPLICATION INSTRUCTIONS

Before applying the brace, please read these instructions completely and carefully.


Correct application is important to the proper functioning of the brace.

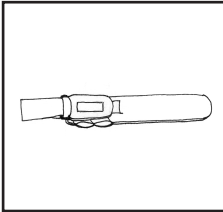


1. Remove strap from packaging, and layout as shown.

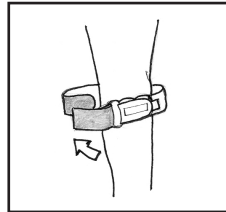

2. Place buttress on affected area, encircle the limb,

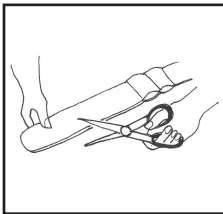

3. Trim away excess strap. Be sure that the trimmed end of the strap still reaches under the plastic buckle, just short of the buttress.

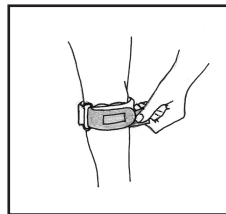

4. Reapply strap with buttress on affected area. Pull logo end through plastic ring to make strap snug fitting.

### USE AND CARE

- Hand wash in cold water using mild soap, rinse thoroughly. **AIR DRY ONLY.** Hinges may be removed for washing. **Note:** If not rinsed thoroughly, residual soap may cause skin irritation and deterioration of the material.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water. **AIR DRY ONLY.**

**WARNING: NEOPRENE SUPPORTS SHOULD NOT BE WORN BY INDIVIDUALS WITH KNOWN SUSCEPTIBILITY TO DERMATITIS OR ALLERGIES TO NEOPRENE BY-PRODUCTS. IF YOU EXPERIENCE ANY PAIN OR SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.**

### WARRANTY

Top Shelf guarantees this product to be free of defects in material and construction for a period of six months from date of purchase.