



1851 East Paradise Road, Suite A
Tracy, CA 95304
Phone: 866.592.0488 • Fax: 209.834.8832
topshelforthopedics.com



Tele-Range

Instructions For Use

Please read these instructions completely and carefully before applying this product . Correct application is important for proper function of the brace.










Tele-Range

Instructions for Use

Please read these instructions completely and carefully before applying this brace. Correct application is important to the functioning of the brace.

Indications: Post-operative recovery daily use.

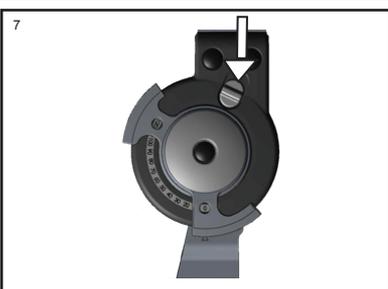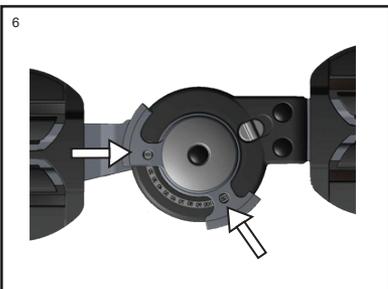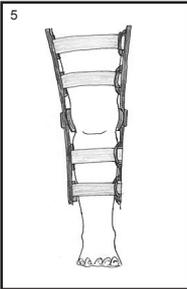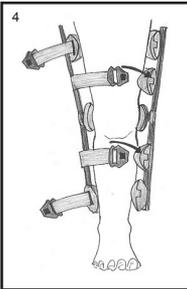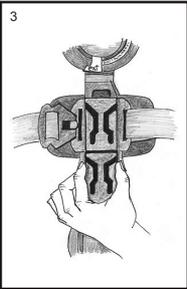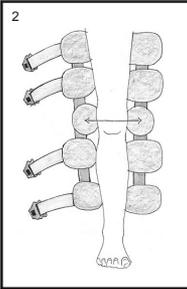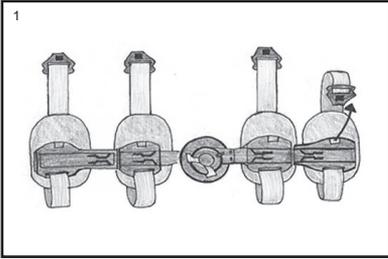

1. Unclip all buckles and fold back the straps just above and below the hinge. Spread the hinges apart.
2. Lay the brace out flat, with the straps behind the leg and a hinge on each side of the leg. Align the center of the hinge with the center of the patella (knee cap).
3. Adjust the length of the thigh and calf sections. Make sure that both hinge bars are equal in length.
4. Align the hinges with midline of leg and secure the straps closest to the knee.
5. Fasten the remaining straps. It may be necessary to retighten the straps closest to the knee. Strap lengths may be adjusted by removing the strap end and folding over or cutting away excess strap material and reaffixing the strap ends.
6. Range of Motion Settings
Range of motion maybe adjusted from - 10° extension and 120° flexion, position the hinge and slide forward the extension lock button to lock the hinge.
7. Extension Lock Settings
To lock the hinge in a desired position between - 0° extension and 40° flexion, position the hinge and slide inward the extension lock button to lock the hinge.

Use and Care:

The liner and straps may be hand washed with a mild detergent. Air dry only. Foam and straps may be trimmed and adjusted, but only at the discretion of your Health Care provider.

Warning:

If you experience pain, swelling or sensation changes as a result of wearing this brace, consult your healthcare provider immediately. All adjustments of the brace should be done under the direction of a healthcare professional.

Warranty:

Top Shelf Orthopedics will replace all or part of the brace for defects in material or workmanship for the period of six months from the date of purchase.

Caution:

Federal Law (USA) restricts this device for sale by or on the order of a licensed healthcare professional.