

topshelforthopedics.com  
Phone: 866.592.0488 • Fax: 209.834.8832  
Tracy, CA 95304  
1851 East Paradise Road, Suite A



# Universal Shoulder Immobilizer

## Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.




# Universal Shoulder Immobilizer

## Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.

Indications: Immobilization & support of the shoulder and elbow joints.

1. Position abdominal belt (A) and secure at front.
2. Secure bicep cuff (B) around arm with attachment straps towards the body.
3. Position shoulder strap (C) opposite affected shoulder, and secure to abdominal belt.
4. Secure wrist/forearm cuff around wrist. Attach bicep cuff to abdominal belt. Adjust as necessary (D).

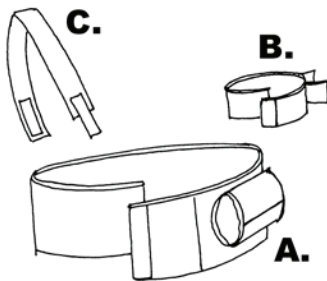

### Use and Care:

Hand wash in cold water using mild soap. Rinse thoroughly, and air dry only. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

### Warning:

This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

### Warranty:

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months from the date of purchase.

### Caution:

Federal Law (U.S.A.) restricts this device to sale by or on the order of a licensed healthcare professional. For single patient use only.